




## 4. Remote control error messages

In case of error during operation, the following messages appear on the remote control display:

Symbol	Symbol	Error location
<b>tH</b>	High Temperature Error	External Projector Internal Projector Main Heating Element Auxiliary Heating Element
<b>SE</b>	Faulty Temperature Sensor	Internal Projector Main Heating Element Auxiliary Heating Element
<b>HE</b>	Heating Element Error	External Projector Internal Projector
<b>PH</b>	High Temperature Error	Internal circuit
<b>Hod</b>	Overloading Motor Error	Horizontal motor drive
<b>HEE</b>	Faulty Motor Encoder	
<b>HLE</b>	Faulty Upper & Lower Limit Switch	
<b>Vod</b>	Overloading Motor Error	Vertical motor drive
<b>VEE</b>	Overloading Motor Error	
<b>CE</b>	Faulty Communication Line	Main computer and remote control connection Internal connection for main computer

Please contact the support center if any of the following symbols appear  
: tH, SE, HE, PH, Hod, HEE, HLE, CE, Vod, VEE

 Caution	Check the error messages displayed and contact the Customer Support Center.
--	---